Vet Expert Rapid Test FCoV Ab

EN PL DE FR ES PT IT CZ LV RO SI
LT HU GR MK UA RU TR AL BG

EN PRINCIPLES
The Vet Expert FCoV Ab is an immunochromatographic test for the qualitative detection of antibodies against Feline Coronavirus in feline serum, plasma or whole blood. Letters T and C on the surface of the test indicate the line of test (T) and control line (C). The control line (C) is used for procedural control and should always appear if the test procedure is being performed properly and the test reagents are working correctly. The test line (T) appears when there are antibodies against FCoV in the sample.

Sensitivity: 96.1%
Specificity: 98.0%

MATERIALS PROVIDED (2/5/10 TESTS/KIT)

- 1) 2/5/10 Vet Expert FCoV Ab test plates
- 2) 1/2 assay diluent bottle (2 ml)
- 3) 2/5/10 anticoagulant tubes
- 4) 2/5/10 disposable pipettes (10 µl)
- 5) Instruction for use

PRECAUTIONS:

- 1) For veterinary diagnostics use only.
- 2) For best results, strict adherence to the instructions is required.
- 3) The test plate is sensitive to humidity as well as heat. Perform the test immediately after opening the pouch.
- 4) The test components are single-use only.
- 5) Do not touch the membrane in the result window.
- 6) Do not use the test kit beyond the stated expiration date marked on the package label.
- 7) Do not use the test kit if the pouch is damaged.
- 8) Do not mix components from different lot numbers.
- 9) Handle with the sample as potentially infectious.
- 10) Decontaminate and dispose of all samples and materials in accordance with national and local regulations.

11) Consider the test results as invalid after the specified read out time.

STORAGE OF THE TEST KIT
The test kit should be stored between 2-30°C. The test is stable within the expiration date. DO NOT FREEZE. Do not expose the test to direct sunlight.

SPECIMEN COLLECTION AND PREPARATION
Serum, plasma or whole blood are appropriate for this test. However, serum is the most advisable sample. [Serum] Collect the whole blood into the collection tube (NOT containing anticoagulants), leave to settle for 30 minutes for blood coagulation and then centrifuge to get serum.

[Plasma] Collect the whole blood into the collection tube (containing anticoagulants) and then centrifuge to get plasma.

[Whole blood] Collect the whole blood into the anticoagulant tube (max. vol. 1.5 ml). If anticoagulated whole-blood is not immediately tested, it should be refrigerated at 2-8°C and used within 24 hours. Serum samples should be stored at 2-8°C. For longer storage, freeze the samples at -20°C or below. Avoid repeated freezing and thawing. Samples containing precipitate may yield inconsistent test results. They must be clarified prior to assaying. Hemolyzed or contaminated samples may give erroneous results.

PROCEDURE OF THE TEST (LOOK AT THE TOP)

All reagents and samples must be at room temperature (15-30°C) before use.

1) Remove the test from the foil pouch and place it on a flat and dry surface.

2) Using a pipette (not included), add 10 µl (marked on the pipette) of serum/plasma/whole blood into the sample well.

3) Dispense 3 drops of assay diluent into the sample well.

4) The sample will flow across the result window. If it does not appear after 1 min, add one more drop of the assay diluent to the sample well. Read the result after 10 min. Do not read after 20 min.

INTERPRETATION OF THE TEST (LOOK AT THE TOP)

1) Negative result – only control line (C) appears in the result window.

2) Positive result – both test (T) and control (C) lines appear in the result window ~ indicates the presence of target antibodies. Even a fade line states a positive result.

3) Invalid result – if the control line (C) does not appear, the result might be considered invalid. The same applies to the test line (T).

LIMITATIONS OF THE TEST

Although the Vet Expert FCoV Ab test is very accurate, a low incidence of false results can occur. In case of questionable results, other tests should be performed. Definitive clinical diagnosis should not be based on the results of a single test, but must be an effect of extended clinical and laboratory analysis.

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Test procedure

1) Add 10 µl of sample to the sample well.

2) Add 10 µl of assay diluent to the diluent well.

3) Wait for 10 minutes for the result to appear.

4) Read the result after 10 minutes. Do not read after 20 minutes.

Test result

1) Negative: only control line (C) appears in the result window.

2) Positive: both test (T) and control (C) lines appear in the result window ~ indicates the presence of target antibodies. Even a fade line states a positive result.

3) Invalid: if the control line (C) does not appear, the result might be considered invalid. The same applies to the test line (T).

Sensitivity: 96.1%
Specificity: 98.0%

MATERIALS PROVIDED (2/5/10 TESTS/KIT)

1) 2/5/10 Vet Expert FCoV Ab test plates

2) 1/2 assay diluent bottle (2 ml)

3) 2/5/10 anticoagulant tubes

4) 2/5/10 disposable pipettes (10 µl)

5) Instruction for use

PRECAUTIONS:

1) For veterinary diagnostics use only.

2) For best results, strict adherence to the instructions is required.

3) The test plate is sensitive to humidity as well as heat. Perform the test immediately after opening the pouch.

4) The test components are single-use only.

5) Do not touch the membrane in the result window.

6) Do not use the test kit beyond the stated expiration date marked on the package label.

7) Do not use the test kit if the pouch is damaged.

8) Do not mix components from different lot numbers.

9) Handle with the sample as potentially infectious.

10) Decontaminate and dispose of all samples and materials in accordance with national and local regulations.

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STORAGE OF THE TEST KIT

The test kit should be stored between 2-30°C. The test is stable within the expiration date. DO NOT FREEZE. Do not expose the test to direct sunlight.

SPECIMEN COLLECTION AND PREPARATION

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