Giardia + CPE combined test

ES Prueba combinada Giardia + CPE

FR Test combiné Giardia + CPE

DE Giardia + CPE Kombitest

IT Test combinato Giardia + CPE

NL Giardia + CPE gecombineerde test

PT Teste combinado de Giardia + CPE

CS Kombinovaný test Giardia + CPE

EN Giardia + CPE combined test
Instructions for use

The Giardia + Clostridium perfringens enterotoxin combination test enables rapid and reliable detection of Giardia and Clostridium perfringens enterotoxin antigens in faecal samples.

Sample treatment

Fresh faeces do not require any special pre-treatment and can be examined directly. The samples can be refrigerated at 2-8°C for up to two days and stored at -20°C for several weeks. Before the test is carried out, the samples must be brought to room temperature. Frozen samples should be thoroughly mixed after thawing.

Test procedure (Figure 1-4)

Figure 1: Take a faecal sample with the cotton swab. Make sure that the upper end of the cotton swab is covered with faeces.

Figure 2: Unscrew the sample container and hold the faeces/cotton swab in it.

Figure 3: Move the cotton swab back and forth in the liquid to loosen the faeces. Then close the sample container and shake so that the sample and liquid mix well. This will discolour the liquid. NB: Please use the mixed sample immediately and carry out the test.

Figure 4: Open the aluminium packaging of the test cassette and place the cassette on a horizontal surface. Break off the upper tip of the sample vial by pressing firmly. Add 3 drops of the sample material to each sample field of the test cassette. If the liquid does not run up after a few seconds, add another drop of sample liquid to the sample field. If the liquid remains in the sample well and is not absorbed, it helps to press lightly on the sample field with the upper end of the cotton swab.

Evaluation (Figure 5)

The test can be analysed after 10 minutes. One or two blue lines appear in the reaction field.

Positive test result (Figure 5 +):

The test line and the control line become visible. Giardia and/or Clostridium perfringens enterotoxin antigens are detected. NB: A weak test line must also be interpreted as a positive test result. An excessive amount of sample faeces can cause a brown line in the T region (T), which must not be interpreted as a positive result. The test is invalid and should be repeated.

Negative test result (Figure 5 -):

Only the control line is visible.

Invalid:

If no control line is visible, the test is invalid.
NB: All test results read later than 10 minutes are invalid.

Disposal:

All used materials must be disposed of carefully as the material may be infectious. Test materials and used cassettes must be packed in plastic bags and then disposed of.

Please note:

For veterinary use only. • For professional use only. • Use cassette as soon as possible after opening. • Please observe the required sample quantity. • Do not apply any sample material to the reaction field. • Do not touch the reaction field. • The pre-coloured lines in the reaction field are used for quality control. The pre-colouring is washed away by the liquid during the test.

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